Simulaids Pediatric ALS trainer is completely Latex Free. All Simulaids products are Made in the USA.

Features:

- * Bag valve Mask Ventilation
- * Intubation Head
- * Oral and Nasal Intubation
- * Sellick Maneuver
- * Accepts NG Tube (Functional)
- * Three lead monitoring
- * Brachial pulse
- * External chest compressions
- * Jaw Thrust
- IV Arm & Hand
- * Two I/O legs for interosseous needle insertion and aspiration of bone marrow
- * Scalp vein for palpation
- * Cranial sutures and Fontanels

Warranty:

Simulaids warrants all ALS products we manufacture against defect caused by materials or workmanship for one year. Simulaids, Inc. is not liable for damage or injury to persons or property which may result from use other than stated use. Electronic parts will carry a 90 day warranty.

Replacement Parts

Item No	Description
091 092	Overlay skin
093 094	I/O Bone and Blood Capsule pk/12
095 098	I/O Leg Skin pk/4
101 102	IV Arm Skin
144 225	IV Vein Set
	Pediatric ALS replacement lung pk/4
	Pulse bulb
	Arrhythmia Simulator
	IV Reservoir (1)
	Powder to make 1 gal. Blood

Pediatric ALS Trainer

Directions for Use
Item #s 101-090/ 101-091

Made in
the USA

LATEX
FREE

SIMULAIDS, INC.

PO BOX 807 12 Dixon Ave. Woodstock, NY 12498

Phone: (914)-679-2475

(800)-431-4310

Fax: (914)-679-8996

www.simulaids.com

Instructions for use:

EKG Monitoring **Connect the three external connectors** to a rhythm simulator box. Simulaids rhythm simulator has color coded connectors on the front of the box. It also features an adjustable heart rate which is controlled by the dial on the right side of the simulator.

Brachial Pulse

Attach bulb to connector at the end of the clear tubing on left arm. Gently compress to activate pulse.

Intubation

Lubricate the ET tube with KY Jelly (supplied). Trainer will accept up to a 3.5 ET tube and can be intubated orally or nasally. ET tube can accept simulated medication. (Water is recommended for this procedure.)

NG Feeding Tube

Lubricate NG tube with KY Jelly (supplied). After placing tube student can simulate feeding. (Water recommended for this procedure.)

Lung and Stomach replacement

Unsnap the overlay. Remove lung/stomach assembly. (Dispose of properly.) Insert new lung/stomach assembly. Snap overlay.

Replacement of I/O Bone

Remove leg skin by gently pulling from the knee towards the foot. Discard old bone and simulated blood capsule properly. Replace new bone and blood capsule. Replace leg skin. (Powder may be needed to ease this process.)

Cleaning

External

Clean In mild soapy water. Rinse thoroughly with clear water. **DO NOT IMMERSE IN WATER!**

Cleaning

Internal

If internal cleaning becomes necessary due to contamination remove lungs and stomach assembly, insert ET tube into trachea and flush with CDC recommended solution. Repeat procedure with ET tube placed in esophagus.

IV Arm & Hand

HOW TO USE

Put a drop of either dish detergent or surgical lubricant onto tip of each connector on reservoir tubes and connect to the silicone tubes of the simulated arm or hand. Make sure tube is pushed up past the ribbed section of the connector to avoid leakage.

Fill reservoir bag with simulated blood or colored water,

Elevate the filled reservoir bag and open the slide clamps of both reservoirs. Gravity will circulate the simulated blood through the venous network and into second reservoir bag

When elevated reservoir is empty, reverse with lower one now filled with fluid.

NOTE: Simulated blood which accumulates under the skin, in the

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