


## Your Kit should include:

- 1. The patented Act+Fast Anti ChokingTrainer
- 2. An internal Air Bladder
- 3. Clear Plastic Airway. (This should fit snuggly into the coupler on the Air Bladder).
- 4. 30 Foam Plugs
- 5. If you have the Red Back Slap Pad model, this should include both a plastic plate and foam pad inside.
- 6. User Manual, please refer to for detailed instructions.
- 7. Nylon drawstring Carry Bag

## Foam Plugs: The plugs can be customized to your student's needs.

- To make them **easier** to shoot out, the plugs can be rolled in your fingers to soften, or trimmed a tiny bit in length.
- To make them more challenging to use, a small piece of tape can be wrapped around.

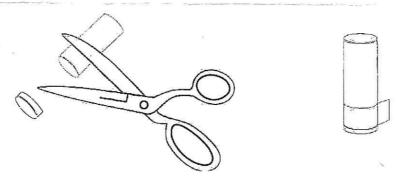

## **IMPORTANT:**

First try the Anti Choking Trainer on yourself to ensure it is working properly before using it on another person. It should not require force to use. The air bladder should be soft enough that proper technique will easily expel a foam plug.

It is important not to be forceful: to avoid injury while training and while trying to rescue a choking victim.

Send any questions: info@actfastmed.com Call Toll Free: 855 934 9340

For warranty information, go to: www.actfastmed.com/faq/